

Decontamination Protocols



Disconnect the Trolley from the electricity supply before carrying out decontamination procedures.



Basic cleaning information

Only use disinfectants designed for cleaning healthcare equipment i.e. Sodium Dichloroisocyanurate (NaDCC), Sodium Hypochlorite (NaOCL) or similar (up to 10,000 ppm available chlorine).



Do not use alkaline or acid chemical products. Do not use abrasives (scouring powder), scourers or other materials / agents which could damage the bed coating. Do not use caustic or corrosive substances to clean the bed. Do not use chemical substances which may affect the finish of the bed, such as acetone, petroleum etc.



It is recommended that only CE marked cleansers & disinfectants are used to clean the beds. shall not be liable for any damages caused using inappropriate detergents or disinfectants.



Do not use phenol based disinfectants.

+
 The Felgains trolley should be cleaned and disinfected as per the 'Cleaning Regimes' table below or in line with local infection control policy: -

Cleaning Regimes

Level of cleaning	Areas to be cleaned
1. Daily Cleaning	All exposed surfaces of the bed: - <ul style="list-style-type: none"> ➤ Head and foot boards ➤ All areas of Stowable Safety Rails ➤ Exposed areas of mattress platform ➤ Accessories (handsets, IV pole, push handles etc.)
2. Clean before every use	Trolley parts stated in (1) and: - <ul style="list-style-type: none"> ➤ All the areas of the mattress platform. The upper parts of the undercarriage. ➤ Connecting conduits between mattress platform and undercarriage. ➤ Freely accessible metal parts of the mattress platform. ➤ Bumper wheels (x4). ➤ Central braking system levers.
3. Complete Cleaning and Disinfection	All parts of the Trolley stated in (1) and (2), and: - <ul style="list-style-type: none"> ➤ All the metalwork under the mattress platform. ➤ All areas under the mattress platform.

Cleaning

Wearing suitable protective clothing, clean all surfaces with a disposable cloth soaked in a neutral detergent and hand hot water, being careful with the beds electrical parts. Start by cleaning the upper sections of the bed and work along all horizontal surfaces. Work methodically towards the lower sections of the bed and clean the wheels last.

Take extra care to clean areas that may trap dust or dirt. Rinse with clean water and dry with disposable paper towels. Allow the cleaned parts to dry before replacing the mattress.

Disinfecting

After cleaning the bed as described above, wipe all surfaces with *Sodium Dichloroisocyanurate (NaDCC)*, *Sodium Hypochlorite (NaOCL)* or similar at a concentration of 1,000 parts per million (0.1%) of available chlorine. Wipe dry with a clean dry cloth. In the case of pooling body fluids, e.g. blood, the concentration of NaDCC or NaOCL should be increased to 10,000 parts per million (1%) of available chlorine. Wipe dry with a clean dry cloth.



Felgains Ltd
33 Knightsdale Road
Ipswich
Suffolk IP1 4JJ

T +44 (0) 1473 741144

F +44 (0) 1473 461933

E mail@felgains.com

felgains.com